## PAVIS S.p.A.

Sede / Headquarter:

Via G. Rossini, 11/B - 21020 Buguggiate (VA) - Italy

Tel. (+39) 0332 45 54 63

Fax (+39) 0332 45 97 48

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#### 1 - INDICATIONS

- Traumatic pathologies: wrist sprains, flexor and extensor tendon injuries, thumb fractures/sprains
- Tendinitis, osteoarthritis, arthritis, De Quervain's disease, carpal tunnel
- Post-traumatic / post-surgical / post-plaster cast "joint rest" (also at night)

#### 2 - CONTRA-INDICATIONS AND SIDE-EFFECTS

The choice between different models must be done according to the pathology and the necessities stated by the doctor. Since this device is NON-INVASIVE and made with non-allergenic fabrics, no negative or side effects have ever been noted. Any serious incident occurred in relation to the device use, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### 3 - MAINTENANCE AND CARE

- Before washing, remove all stays and fasten hook and loop close parts

This device is non-biodegradable. Even though it is made of natural fabrics, after use, avoid release the product to the environment. AT THE END OF ITS USABLE LIFE HAS TO BE DISCHARGED ALONG WITH SOLID ORDINARY WASTE.

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