PAVIS S.p.A.

Sede / Headquarter:
Via G. Rossini, 11/B - 21020 Buguggiate (VA) - Italy
Tel. (+39) 0332 45 54 63
Fax (+39) 0332 45 97 48

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UK
1 - INDICATIONS
• Traumatic pathologies: distortions (medial / lateral), tendonitis and sprains
• Rehabilitation phase: passive support to functional rehab of the ankle
• Good support during sport

2 - CONTRA-INDICATIONS AND SIDE-EFFECTS
In case the brace is not correctly fixed, skin redness may arise due to frictions. It is therefore recommended to wear it over a sock.
Since this device is NON-INVASIVE and made with non-allergenic fabrics, no negative or side effects have ever been noted.
Any serious incident occurred in relation to the device use, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

3 - MAINTENANCE AND CARE
• Before washing, hook and loop parts
Washing does not damage the product and it actually restores the elasticity

This device is non-biodegradable. Even though it is made of natural fabrics, after use, avoid release the product to the environment. AT THE END OF ITS USABLE LIFE HAS TO BE DISCHARGED ALONG WITH SOLID ORDINARY WASTE.

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